emmi[®] ULTRASONIC

emmi[®] DENT **emmi**[®] PET

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**READ
CAREFULLY!**

**QUICKGUIDE +
PROBLEM FIXING**

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The device is working correctly.

Red LED:

The device needs to be charged or the attachment is defective.

Replace attachments **every 3 months**.

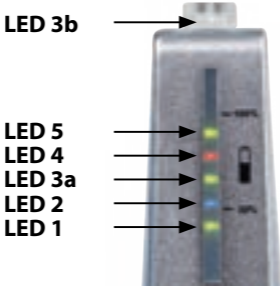
Video tutorial:

Correct placement of the ultrasound attachments for optimal ultrasound transmission!

Scan me

Functions of the LEDs:

a) Charging control function LED2

Blue LED: emmi®-dent is on the charging station

Light blue light: emmi®-dent in charging process

Weak blue light: emmi®-dent is fully charged

b) Battery status

(LED1, LED3a, LED5 & LED3b)

The three **green** LEDs (and the white LED on the brush head) are used in the manner of a sequential light during the cleaning process.

Is the battery fully charged, the LEDs light up as follows:

- LED1, LED3a, LED5 + LED3b

If the battery is approx. 1/3 discharged, the following LEDs light up in sequence:

- LED1, LED3a + LED3b

If the battery is approx. 2/3 discharged, only the following LED lights up:

- LED1

c) „Urgent recharge“ indicator

If the battery is heavily discharged, the LED4 flashes **red** a few times after the cleaning process. Please charge the emmi®-dent.

d) Switching between vibration and acoustic signal (except emmi®-dent Platinum Care)

If the On button of the emmi®-dent is pressed for approx. **10 seconds**, the LED2 flashes **red** and the vibration as well as the acoustic signal are switched off.

Press and hold down the On button for approx. **10 seconds**, and the LED2 flashes **green**. The vibration as well as the acoustic signal are switched on now.

Before first use charge for **24 hours**.

Insert the attachment **vertically**.

Green LEDs:

The device is working correctly.

Red LED:

The device needs to be charged or the attachment is defective.

Replace attachments **every 3 months**.

Video tutorial:

Correct placement of the ultrasound attachments for optimal ultrasound transmission!

Scan me

Functions of the LEDs:

a) Charging control function LED2

Blue LED: emmi®-dent is on the charging station

Light blue light: emmi®-dent in charging process

Weak blue light: emmi®-dent is fully charged

b) Battery status (LED1 & LED2)

The two LEDs are used as battery status indicators during the cleaning process.

If the battery is between full and approx. 1/3 discharged:

- LED1 lights up permanently **green** and LED2 once approx. every 3 seconds

If the battery is approx. 2/3 discharged, only the following LED lights up:

- LED2 lights permanently **green**

c) „Urgent recharge“ indicator

If the battery is heavily discharged, the LED1 flashes **red** a few times after the cleaning process. Please charge the emmi®-dent.

d) Switching between vibration and acoustic signal (except emmi®-pet)

If the On button of the emmi®-dent is pressed for approx. **10 seconds**, the LED2 flashes **red** and the vibration as well as the acoustic signal are switched off.

Press and hold down the On button for approx. **10 seconds**, and the LED2 flashes **green**. The vibration as well as the acoustic signal are switched on now.

Please consider:

emmi®-pet is completely soundless and vibrationless.

Here you will find independently correctable errors that could occur with your emmi®-device:

1) Flashing of the red LED means:

a) Battery discharged:

Place the handpiece on the charging station so the emmi®-ultrasound device can start the charging process.

b) Indication of a defective attachment:

Place a new attachment on the handpiece and switch the device on again. The **green LED** should now light up continuously.

2) Green LED lights up and turns off immediately:

a) No attachment attached: Please attach an attachment to the handpiece and switch it on again. The **green LED** should now light up continuously.

b) Attachment defective: Please attach a new attachment to the emmi®-ultrasonic device and try to switch it on afterwards. The **green LED** should now light up continuously.

3) The green LED lights up continuously, but without vibration:

Vibration was manually switched off:

Please place an attachment on your emmi®-device. Press and hold the **ON/OFF key** for approx.

10 seconds to switch on. The LED lights up **green three times**. You can now remove your finger from the ON/OFF key. After switching the device on, the vibration should work again.

To **switch off** the vibration, please proceed in the same way. The LED will light up **red three times**.

(Please note that our models emmi®-dent Platinum Care and emmi®-pet do not have a vibration motor and therefore this function cannot be used with these models).

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E-Mail: info@emmi-ultrasonic.de

You need personal assistance?

Please feel free to contact our customer service:

Phone: +49 (0) 6105 / 40 67 94

WhatsApp: +49 (0) 160 / 50 91 446

E-Mail: info@emmi-ultrasonic.de

Emmi Ultrasonic GmbH

Gerauer Str. 34 | 64546 Moerfelden-Walldorf

Tel.: +49 (0) 6105 / 40 67 94 | info@emmi-ultrasonic.de